---

**Iris**  
**Iris Plus**

---

**Français - Mode d'emploi - Page 1**  
**Nederlands – Gebruikshandleiding - Pagina 7**  
**English - User Manual - Page 13**  
**Deutsch – Gebrauchsanweisung - Seite 19**  
**Specifications 25**

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This manual describes the KOBA Vision Iris and Iris Plus. The latest versions of the manual can be downloaded from the KOBA Vision website: [www.kobavision.be](http://www.kobavision.be).

Release date: December 2019.

## Chapter 1: Safety and maintenance instructions

### SAFETY INSTRUCTIONS

- The device is not to be used in a medical environment.
- Never sit on the sliding table.
- Never attempt to pull yourself up by holding the Iris .
- The power supply must only be replaced by the same type (15V, max. 2,5A, EN60601-1) after switching off the device and removing the adapter from the grid.
- The connections have been designed to only connect in 1 way. Never force a connection. The cables are only to be connected by authorised persons. Install the power supply of the device in a well ventilated area.
- Never attempt to repair the device by yourself. It is dangerous to remove protective covers which are there to prevent electric shock. You must contact KOBA Vision or your dealer in case of defects.
- To avoid electrocution, do not place the device in a spot where it can come into contact with liquids.
- When moving the device, the height adjustment must always be blocked by using the supplied orange screws.
- Consult the user manual of the screen for specific safety instructions related to the screen.
- Save this manual carefully. It contains all necessary information about your device.
- Keep the device in a dry, well-ventilated and clean place. Do not expose the device to direct sunlight or a heat source.
- Never use this device in ambient temperatures higher than 35°C.
- This device is designed as aid to magnify and to read aloud documents. Any change of purpose voids the CE-marking.
- Your product is designed and manufactured with high quality materials and components, which can be recycled and reused. Do not dispose of your old product in your general household waste bin. Dispose of the complete product (including its cables, plugs and accessories) in the designated collection facilities.


# WARNING

The screen is to be assembled and disassembled by authorized persons only. Incorrect installation or assembly can cause severe damage and/or injuries.

You must be extremely cautious when moving the Iris .  
The Iris must not be picked up by the screen, the front panel, the light casings or the slide table. Always block the height adjustment with the supplied orange screws.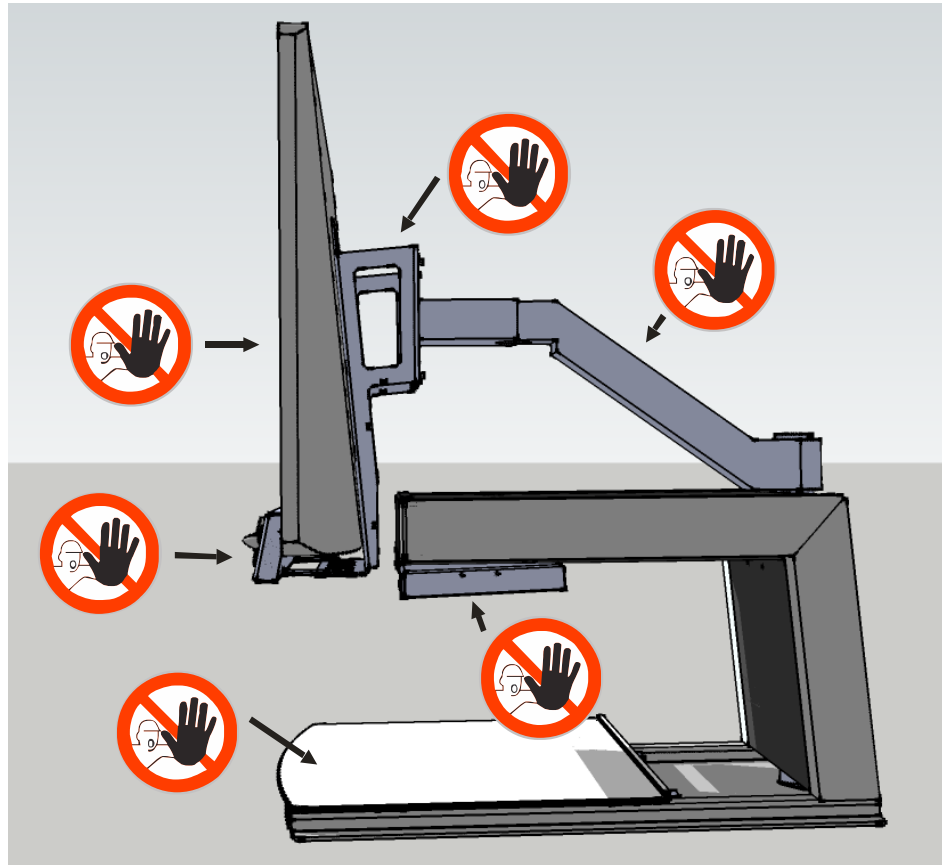

## MAINTENANCE INSTRUCTIONS


- Occasionally clean the screen and the lens using a damp cloth. For further instructions: check the screen manual.
- Treat your device with care and you will enjoy it for years to come.

## *Chapter 2 : Installation*

The Iris is solely to be installed by authorised personnel. You can find a list on [www.kobavision.be](http://www.kobavision.be)

# Chapter 3: Controls

## OPERATING PANEL

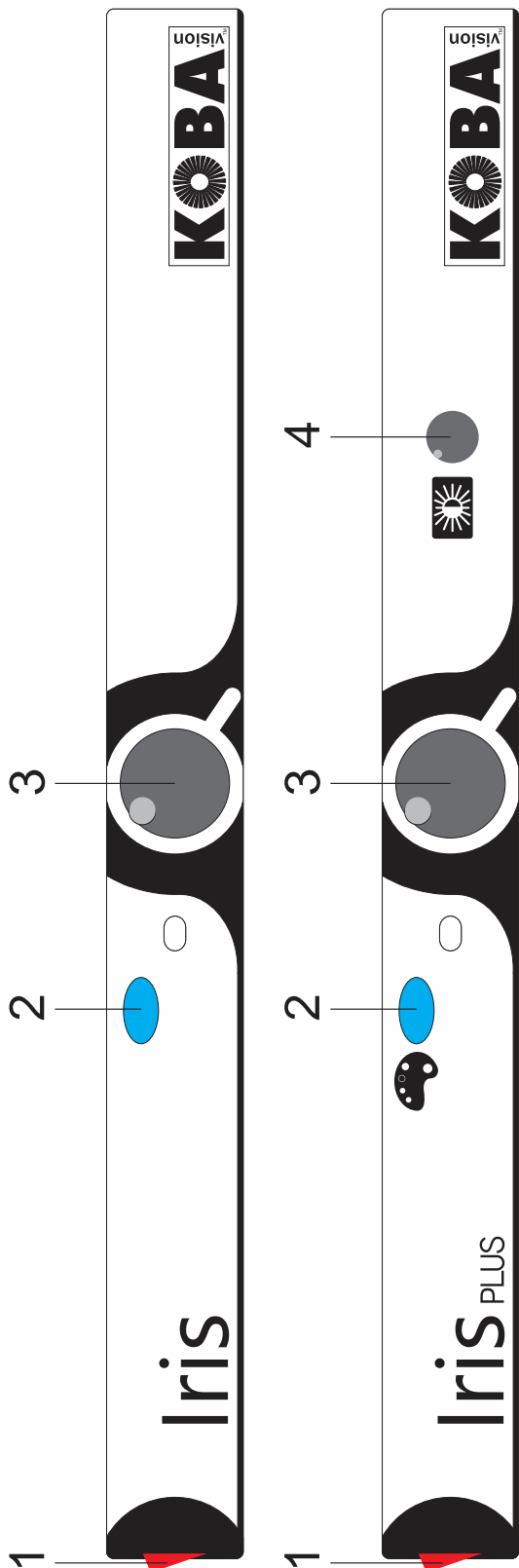1	ON/OFF switch
2	<b>Press briefly:</b> switch between photo mode, negative, positive reading <b>Hold button:</b> changes colour of the background or characters
3	<b>Turn:</b> zoom in/zoom out <b>Press:</b> overview/magnification
4	<b>Turn in colour mode:</b> brightness <b>Turn in positive or negative reading :</b> improve background


# SWITCHING ON/OFF

The device is switched on with **button 1**. Most screens are switched on by the Iris . If this is not the case, switch on the screen as well (see screen manual).

The Iris is switched off with **button 1**. Some screens switch off the Iris when they are switched off.

After switching on, an on-screen bar shows the start-up progress. The Iris indicates when it is ready to be used by telling “the device is ready for use”.

# SLIDING TABLE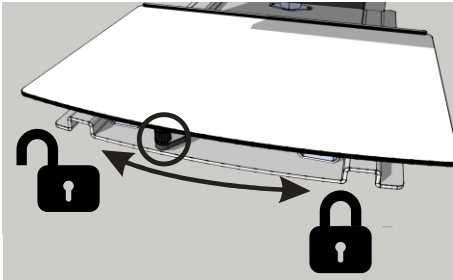


Under the table you can find a lever which controls the brake. If you move the lever to the left, the slide table is free to move left/right/up/down. If you move the lever to the right you adjust the friction. If you move the lever all the way to the right, the slide table will be blocked.

# HEIGHT AND TILT ADJUSTMENT

## Height (Flex arm and large screens)

According to personal preference, the screen of the Vocatex can be moved up or down. To do this, make sure you take hold of the screen on both the left and right side of the screen.

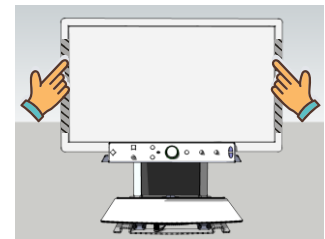
### Increase Lift Strength (Flex arm)

If the screen doesn't stay up when raised, increase the lift strength by turning the angle key right.

### Decrease Lift Strength (Flex arm)

If the screen doesn't stay down when lowered, decrease the lift strength by turning the angle key left.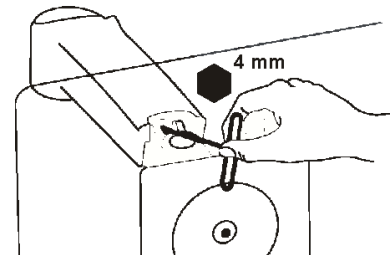


## Tilt (Flex arm)

According to personal preference, the screen of the Iris can be tilted. To do this, make sure you take hold of the screen on both the left and right side of the screen.

### Increase Friction

If the screen tilts down, then you'll need to increase friction by turning the angle key right.

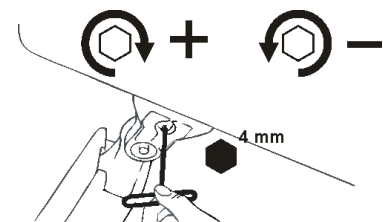## Decrease Friction

If the screen is too difficult to tilt, then you'll need to decrease friction by turning the angle key left.

## USAGE TIPS

### Reading

Reading is best done in positive or negative mode. These modes adapt the image for the visually impaired to a high-contrast image with improved background. Background and letters can be set at your discretion. The background can be set by means of background suppression (Quartz). This will allow you to filter out the structure of a sheet. This unique KOBA Vision function allows you at all times to set a high contrast background without annoying paper impurities. A newspaper can be read without annoying paper impurities. Factory settings will take you back to the original settings.

### Watch photos

Photos are best watched in photo mode. This mode is a unprocessed image without contrast enhancement. By changing the brightness (Quartz), you can adjust the image so that the photo becomes clearly visible.

### Crafting

Crafting is best done with a small magnification. It is better to select a dark background during this activity so that the camera is not hindered by large differences in contrast with whatever is darkened by the hands.

### Writing

You can also write under your video magnifier. It is best to use a black pen and lined paper.

## ***Chapter 4: Troubleshooting***

Below is a list of problems that may occur with your Iris . Consult the list first before contacting KOBA Vision or your dealer.

<b>Symptom</b>	<b>Symptom</b>	<b>Symptom</b>
The device isn't turning on	No display on the screen	No sound
<b>Possible causes</b>	<b>Possible causes</b>	<b>Possible causes</b>
No power	Power failure The screen is not switched on	The screen volume is on zero The sound has been disrupted The headphone is plugged in The HDMI connector is loose
<b>What to do</b>	<b>What to do</b>	<b>What to do</b>
Check the power cable	Check whether the plug of the power cord has been properly plugged into	Slightly turn up the screen volume

<p>Check the adapter and verify if a green led is burning</p>	<p>the wall socket If the device still won't work, pull the plug out of the wall socket. Then wait 60 seconds before plugging it back into the wall socket. Turn the screen on again.</p> <p>Press the "on" button on the remote</p>	<p>Check whether the sound has not been disrupted.</p> <p>Point the remote control directly towards the sensor at the front of the screen.</p> <p>Unplug the headphone</p> <p>Check the HDMI plug.</p>
---	--	--

## Chapter 5: Warranty

### WARRANTY

KOBA Vision offers a two-year warranty on every KOBA VISION video magnifier, starting from the delivery date. To be able to benefit from the warranty, you need to keep the box and the packaging material.

Warranty services are provided by KOBA Vision or dealers acting on behalf of KOBA Vision. When the video magnifier cannot be repaired correctly, KOBA Vision reserves the right to replace the reading device by an equal device. All components and products replaced within the warranty period become property of KOBA Vision.

### WARRANTY PROVISIONS

The warranty becomes invalid if the video magnifier:

- Was not used according to instructions or was misused;
- Was damaged as a result of a fall or impact;
- Was disassembled or tinkered with;
- Was repaired earlier by a non-official service partner.

Consequential loss is under no circumstances covered by the warranty.

KOBA Vision will neither assume any liability whatsoever nor accept any possibility of liability except for the warranty provisions explicitly stated above. This limitation to the liability does not imply a breach of the customer's legal consumer rights or his or her rights vis-à-vis the vendor.

### SERVICE FULFILMENT

Before claiming the warranty or service and before contacting KOBA Vision or a dealer, you must note product name, serial number, purchase date, invoice number and any error messages or particular problem aspects.

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